


Compatibility ACS® MB SC tibial component

In order to standardize **ACS® MB SC** and **MUTARS® GenuX® MK** systems, the design of the **ACS® MB SC** tibial component was adapted. For this purpose, the contour of the posterior recess was adapted to that of the **MUTARS® GenuX® MK**. **ACS® MB SC** tibial components with the new contour are only compatible with MK tibial spacers. There is no compatibility with the **ACS® MB SC** tibial spacers.

Therefore, before implanting an **ACS® MB SC** tibial component with tibial spacers, make sure that you have the correct implants available. Always ensure that you have the implants listed in the surgical technique appendix. Only the implants within a surgical technique are compatible with each other.


ACS® MB SC tibial component
old contour

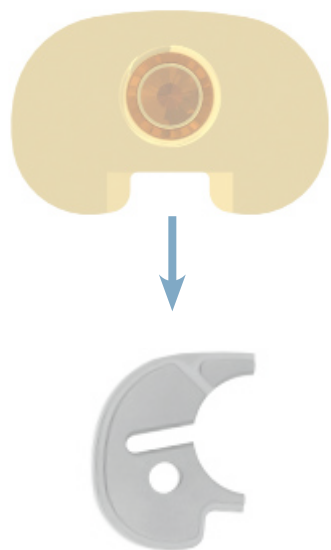

ACS® MB SC tibial spacer

ACS® MB SC tibial component
new contour

after design adjustment

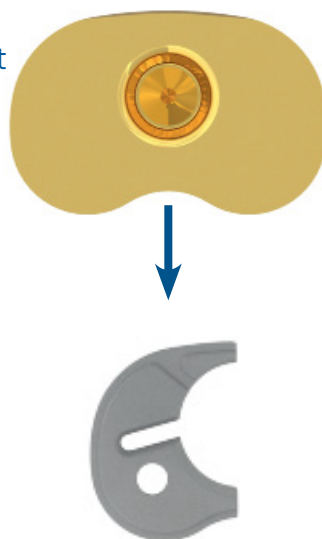

MK tibial spacer

ACS® MB SC tibial component

size	REF
2	4201-4402
3	4201-4403
4	4201-4404
5	4201-4405
6	4201-4406

MK tibial spacer

size	ll/rm		rl/lm	
	5 mm	10 mm	5 mm	10 mm
2	5740-5052	5740-5102	5741-0052	5741-0102
3	5740-5053	5740-5103	5741-0053	5741-0103
4	5740-5054	5740-5104	5741-0054	5741-0104
5	5740-5055	5740-5105	5741-0055	5741-0105
6	5740-5056	5740-5106	5741-0056	5741-0106

walk with us

implantcast GmbH

Lüneburger Schanze 26
21614 Buxtehude
Deutschland

Tel.: +49 4161 744-0
Fax: +49 4161 744-200

info@implantcast.de
www.implantcast.de

CE 0482

